

FCoV Ab

Pagrindinė Informacija

Tikslas: Kačių koronaviruso antikūnų aptikimas

Principas : Imunochromatografinis tyrimas

Mėginys: Kraujas, plazma arba serumas.

Suteiktos medžiagos :

- Anigen Rapid FCoV Ab testavimo prietaisas
- Tyrimo skiedikliai lašinamas buteliukas
- Antikoagulantų vamzdelis
- Vienkartinis kapiliarinis vamzdelis

Ypatumai :

Vieno etapo testavimo procedūra: greita ir tiksli

• Didelis jautrumas ir specifiškumas –

• Jautrumas: 96,0 %, palyginti su IFA


- Specifiškumas: 97,9 %, palyginti su IFA



◆ How to do FIP diagnosis with Rapid FCoV Ab test

- Many healthy cats and cats with conditions other than FIP are FCoV Ab positive.
- Some cats with effusive FIP appear to have low titers or are FCoV Ab negative.
- FCoV Ab testing should only be performed to determine FIP in conjunction with a compatible history, clinical signs, and examination of effusions or blood for high globulins and a low A:G ratio.

Test Procedure

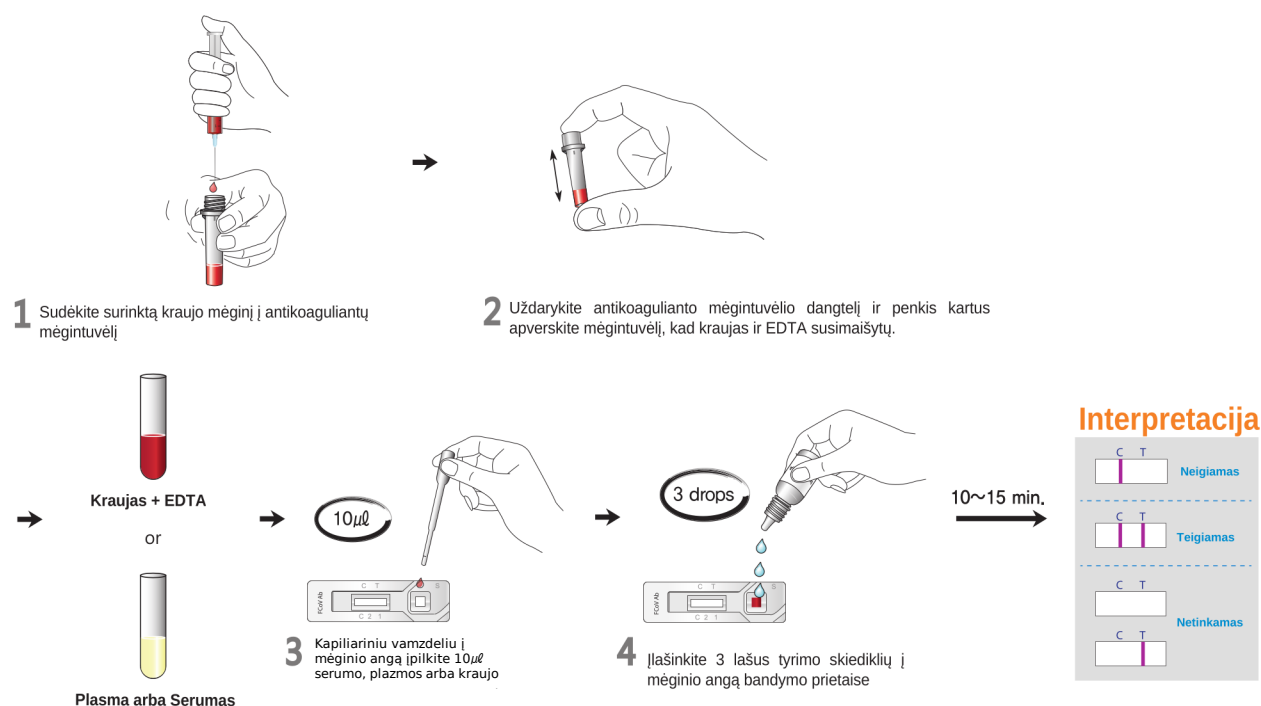

Užsakymo Informacija

Cat. No.	Description	Type	Pack size
RB22-03	FCoV Ab	Device	10 Tests/Kit